



4 EASY STEPS TO CREATE YOUR ITSOCLEAR CLASP IN UNDER 4 MINUTES



Warm clasp by lightly waving it over flame. (WARNING: DO NOT OVER HEAT THE CLASP)



Lightly pass 3 prong plier over flame. (WARNING: OVER HEATING PLIER WILL CAUSE INDENTATION IN CLASP)



Adapt accordingly around tooth by rearming the 3 prong plier as needed.



After clasp is formed around tooth begin to create retention.



Form retention by bending a loop at opposite end of clasp to hold in acrylic.



Once retention is formed the clasp is ready to be set in place on frame.



Secure clasp on frame by using glue and accelerator.



Finally, place acrylic under retentive area and above the clasp so that the clasp will remain in position.



Your partial denture is now ready to be processed as normal.

REPAIRS OR REPLACEMENTS:
Cut groove where clasp is to be formed and cut a deep groove in retentive area. Process as normal.